

ERA[®] is a diagnostic test patented by Igenomix in 2009



3 in every 10 implantation failure patients have a displaced window of implantation*

ERA analyzes 248 genes related to endometrial receptivity

Why choose Igenomix ERA test?



- **ERA is backed by 21 publications** 13 from Igenomix and 8 external publications.



- Randomized study in progress to assess its **applicability in patients without any prior ART.**

(ClinicalTrials.gov Identifier: NCT01954758)



- **Algorithms based on machine learning technologies.**




- **ERA allows clinicians to identify transition phases with 12 hours shifts.**

ERA[®] Results

How to perform ERA biopsy at HRT cycle

- Days with ESTRADIOL VALERATE
- Days with PROGESTERONE

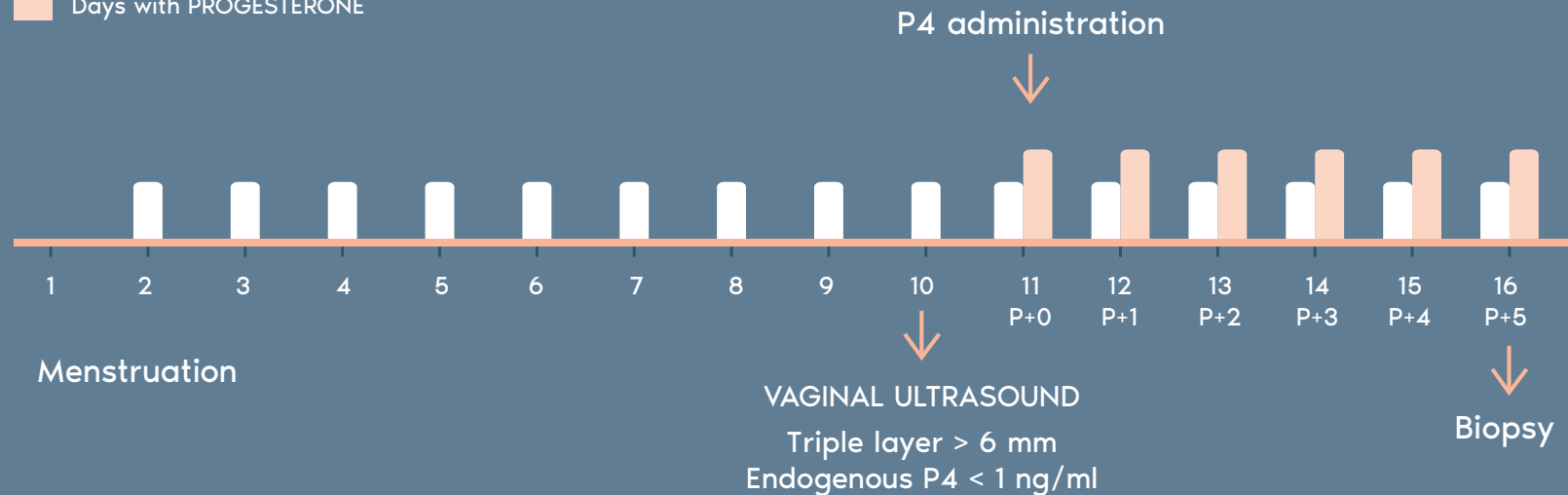

When is a second biopsy needed?

90 % of cases don't need a second biopsy. The second biopsy is only required under two circumstances:

- Patients with a "Pre-receptive" result, that need 2 days more on progesterone.
- Patients with a "Post-receptive" result.

IGENOMIX DATA

